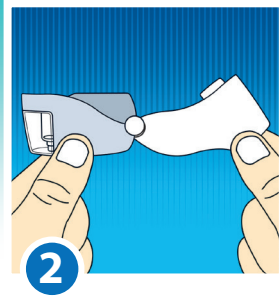
1 Open the Prohaler device by holding the grey part of the device (fixed part) with one hand while pulling back the white part upward (moveable part) gently.

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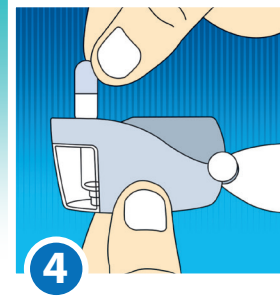
2 Make sure that the Prohaler opens completely.

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3 Remove the capsule from the blister. Ensure that the capsule is used immediately.

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4 Place the capsule inside the capsule chamber.

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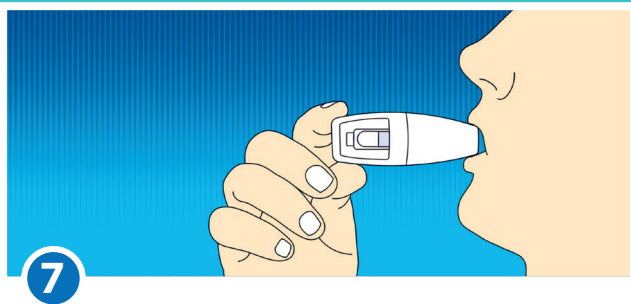
5 Close the Prohaler (as shown in picture). A click sound will be heard indicating proper locking of the device.

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6 Breathe out (exhale) fully. Do not exhale into the Prohaler mouthpiece.

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7 Tilt your head back slightly. Place the mouthpiece in your mouth and close your lips firmly around it. Breathe in quickly and deeply through the Prohaler. You should hear a unique sound caused by the capsule spinning around in the chamber.

(If you do not hear the sound, it may indicate that the capsule has been stuck. In this case, open Prohaler and remove the capsule and then place it again so that it can easily spin around. Repeat steps 6 and 7 to get your complete dose.)

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8 Remove Prohaler from your mouth. Continue to hold your breath as long as you can and then exhale.

(Note: Open Prohaler to check for leftover powder in the capsule chamber. If any remains, then repeat steps 6-8.)

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9 Rinse your mouth with water after use. Remove and discard the empty capsule and close the mouthpiece. Do not leave a used capsule in the chamber.

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10 You must clean Prohaler at least once every week. Rinse the mouthpiece and capsule chamber in running water, remove excess water and leave it to air dry for at least 6 hours.

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